



Surge RX Insert for Apple Vision Pro (Installation Instructions)

Thank you for choosing your custom prescription RX Insert. Follow the steps below to properly install the RX Insert into your Apple Vision Pro eyewear for a secure fit and optimal visual performance.

STEP 1: UNBOX THE RX INSERT PACKAGE

Open the prescription RX insert package and locate the two inserts (left and right) along with the cleaning cloth.



STEP 2: REMOVE THE LIGHT SEAL

On the back of the Vision Pro, detach the magnetic light seal by pulling it gently away from the device.



STEP 3: CLEAN THE LENSES

Using the included cleaning cloth, gently wipe each lens on the Vision Pro to remove any dust or smudges before attaching the inserts.



STEP 4: IDENTIFY LEFT AND RIGHT INSERTS

Remove the inserts from their packaging and check the labeling to identify which insert is for the left eye and which is for the right.



STEP 5: SNAP INSERTS ONTO EACH LENS

Align each insert with its corresponding lens and press gently until it snaps securely into place. The inserts attach magnetically, you'll feel a click when seated correctly.



REMOVING THE RX INSERT

- Look at the inner lens area of the Apple Vision Pro and find the outer edge of the prescription insert where it meets the lens housing.
- Using your fingertip or thumbnail, carefully apply light pressure to the edge of the insert and gently pop it away from the lens. Work slowly. Do not force it or use sharp objects.
- Place both inserts back into their protective case or original packaging to keep them safe and scratch-free.