



Surge RX Insert for Diving Mask (Installation Instructions)

Thank you for choosing your custom prescription RX Insert. Follow the steps below to properly install the RX Insert into your Diving Mask eyewear for a secure fit and optimal visual performance.

STEP 1: CENTER THE RX INSERT

Position the RX Insert in the center of the mask.



STEP 2: ALIGN THE SUCTION CUPS

Align the suction cups on both ends of the RX Insert with the inside surface of the mask lens.



STEP 3: SECURE BOTH SIDES

Press the left and right suction cup into place and make sure the insert is straight.



STEP 4: CHECK THE FIT

Firmly press both ends to secure the insert in place.



REMOVING THE RX INSERT

- Gently take pull one of the suction cup. and do same on the other side.