



Surge RX Insert for Oakley Encoder Strike (Installation Instructions)

Thank you for choosing your custom prescription RX Insert. Follow the steps below to properly install the RX Insert into your Oakley Encoder Strike eyewear for a secure fit and optimal visual performance.

STEP 1: REMOVE THE NOSE PAD

Grip the nosepad firmly and pull it downward from the shield lens to release it from the mounting slot.



STEP 2: INSTALL THE REPLACEMENT NOSE PAD GRIP

Align the nosepad grip supplied in kit on the hooks and press it to insert.



STEP 3: VERIFY ITS IN CENTRE

Hold the RX Insert in front of the nosepad. Slide the RX Insert downward onto the nosepad.



STEP 4: VERIFY THE FIT

Check that the insert is centered and fully engaged with the nose pad assembly.



REMOVING THE RX INSERT

- Gently pull the RX insert upward.
- Grip the nose pad firmly and pull it out of the shield lens and the Shield hook to release the Nose Pad.
- Reinstall the original Nosepad. Reverse of Step 1.