



Surge RX Insert for Meta Quest 3s (Installation Instructions)

Thank you for choosing your custom prescription RX Insert. Follow the steps below to properly install the RX Insert into your Meta Quest 3s eyewear for a secure fit and optimal visual performance.

STEP 1: OPEN THE PACKAGE

Open the prescription insert package and lay out all contents on a clean, flat surface.



STEP 2: CLEAN THE QUEST 3S LENSES

Using the microfiber cloth included in the package, gently wipe the inner lenses of the Meta Quest 3s. Use light pressure. Do not press too hard.



STEP 3: ALIGN AND ATTACH THE FIRST INSERT

Hold the first insert up to its matching Quest 3s lens. Carefully align it so it sits centered over the lens, then gently press it into place. Gently press until it clicks into place.



STEP 4: ATTACH THE SECOND INSERT

Repeat the same process on the other side — align the second insert over its matching lens and gently press it into place.



STEP 5: CHECK THAT BOTH INSERTS ARE SECURE

Lightly wiggle each insert to confirm neither one is loose or shifting. Both should feel firm and stable with no movement before you put the headset on.



REMOVING THE RX INSERT

- Look at the inner lens area of the Meta Quest 3s and find the outer edge of the prescription insert where it meets the lens housing.
- Using your fingertip or thumbnail, carefully apply light pressure to the edge of the insert and gently pop it away from the lens. Work slowly. Do not force it or use sharp objects.
- Place both inserts back into their protective case or original packaging to keep them safe and scratch-free.