



RX Insert for Oakley Meta Vanguard (Installation Instructions)

Thank you for choosing your custom prescription RX Insert. Follow the steps below to properly install the RX Insert into your Meta Oakley Vanguard eyewear for a secure fit and optimal visual performance.

STEP 1: ALIGN THE INSERT

Position the RX insert behind the Oakley Vanguard glasses and align its center clip with the top of the wicket.



STEP 2: PUSH THE INSERT UPWARD

Hold the insert from the bottom and gently push it upward onto the wicket.



STEP 3: SECURE THE INSERT

Check that the insert is centered and firmly attached.



RELEASE THE INSERT

Gently push the RX insert downward from the frame to loosen it from the wicket.



REMOVE THE INSERT

Once it is loose, carefully pull the insert away from the glasses.

