



Surge RX Insert for Motocross Goggle (Installation Instructions)

Thank you for choosing your custom prescription RX Insert. Follow the steps below to properly install the RX Insert into your Motocross Goggle eyewear for a secure fit and optimal visual performance.

STEP 1: INSERT THE BOTTOM TABS

Hold the goggles and place the insert inside while place the two bottom tabs of the insert behind the lower foam, near the nose area.



STEP 2: INSERT THE TOP TABS

Gently lift the top foam and slide top tabs underneath it.



STEP 3: CHECK TABS ARE SECURED

Make sure all four antenna tabs are positioned correctly under the foam



STEP 4: CHECK THE POSITION AND FIT

Make sure the insert is straight, centered, and securely in place.



REMOVING THE RX INSERT

- Gently pull back the top foam to expose the two upper tabs. Carefully release the upper tabs from the foam, one side at a time. Reverse of step 2.
- Lift the RX Insert slightly and slide the two bottom tabs out from the lower foam near the nose area. Reverse of Step 1.