




Surge RX Insert for Oakley EV Zero (Installation Instructions)

Thank you for choosing your custom prescription RX Insert. Follow the steps below to properly install the RX Insert into your Oakley EV Zero eyewear for a secure fit and optimal visual performance.

STEP 1: REMOVE THE NOSE PAD

Grip the nosepad firmly and pull it downward from the shield lens to release it from the mounting slot.

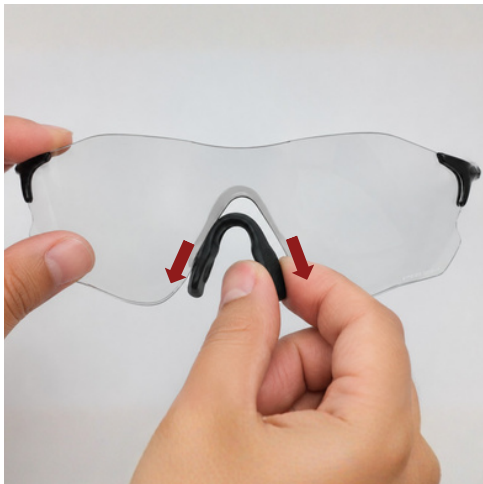

STEP 2: INSTALL THE REPLACEMENT NOSE PAD GRIP

Align the nosepad grip supplied in kit on the hooks and press it to insert.



STEP 3: VERIFY THE FIT

Hold the RX Insert in front of the nosepad. Slide the RX Insert downward onto the nosepad.



STEP 4: REMOVE THE NOSE PAD

Check that the insert is centered and fully engaged with the nose pad assembly.



REMOVING THE RX INSERT

- Gently pull the RX insert upward and away from the nosepad.
- Grip the nose pad firmly and pull it downward from the shield lens to release Nosepad. Step 1