




Surge RX Insert for Oakley Jawbreaker (Installation Instructions)

Thank you for choosing your custom prescription RX Insert. Follow the steps below to properly install the RX Insert into your Oakley Jawbreaker eyewear for a secure fit and optimal visual performance.

STEP 1: UNLOCK SHIELD

Press the nose pads inward using your thumb to lift them up and unlock the wicket. Then, pull down the wicket to release the shield.

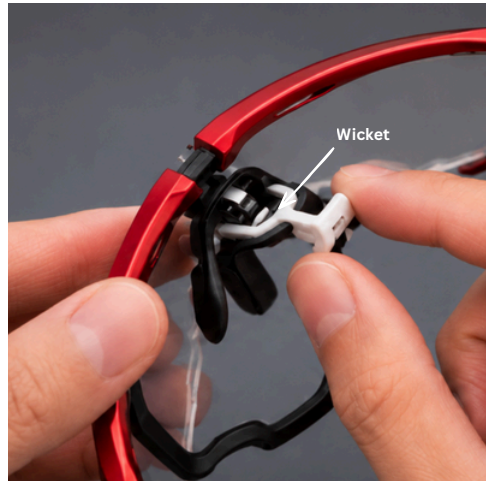

STEP 2: REMOVE SHIELD

Gently open the frame to disengage the shield, then lift the shield out of the frame.



STEP 3: ALIGN AND SEAT THE RX INSERT

Locate the wicket and align the RX insert with the designated mounting point. Gently flex the insert to ensure it is properly seated.



STEP 4: RE-INSTALL THE SHIELD

Attach the shield to the top of the frame, starting from one side and gently flexing it across until the opposite side is seated.



STEP 5: LOCK THE SHIELD

Lock the wicket into the top of the frame to secure the shield, then reposition the nose pad assembly to fully lock the shield in place.



REMOVING THE RX Insert

- Release the Wicket from the nose bridge. Step 1 & 2.
- Gently lift the frame and remove the RX Insert by pulling it away from the Wicket to disengage the RX Insert clip.
- Reinstall the shield by following Step 4 & 5.