



Surge RX Insert for Oakley Radar (Installation Instructions)

Thank you for choosing your custom prescription RX Insert. Follow the steps below to properly install the RX Insert into your Oakley Radar eyewear for a secure fit and optimal visual performance.

STEP 1: DISENGAGE SHIELD

Press the nose pads inward with your thumb



STEP 2: REMOVE SHIELD

Hold the frame securely and pull the shield downward to release it from the frame.



STEP 3: ALIGN AND SEAT THE RX INSERT

Locate the center attachment clip on the RX Insert and align it with the mounting point located on the nose bridge.




STEP 4: REINSTALL SHIELD

RX Insert the right edge of the shield into the right side of the frame. Gently flex the shield and guide the left edge into the left side of the frame until it snaps securely into place.



STEP 5: REINSTALL SHIELD

Push the top center of the shield upward until it clicks into place, securing the shield firmly within the frame.

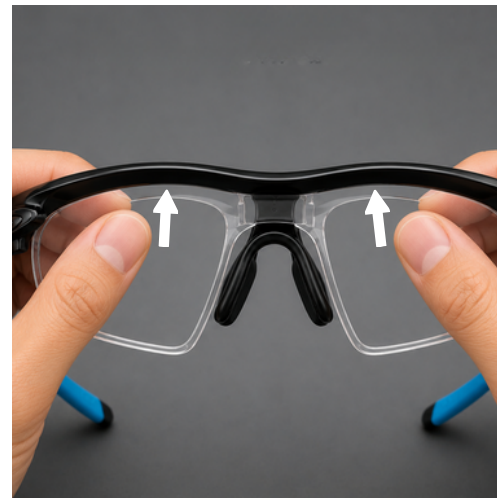

REMOVING THE RX INSERT

- Remove the shield by following Steps 1 & 2.
- Lift the RX Insert away from the nosepad to disengage the RX Insert.
- Reinstall the shield by following Step 4 & 5.