



Surge RX Insert for Oakley Sutro (Installation Instructions)

Thank you for choosing your custom prescription RX Insert. Follow the steps below to properly install the RX Insert into your Oakley Sutro eyewear for a secure fit and optimal visual performance.

STEP 1: DISENGAGE SHIELD

Place your thumbs on the inside of the shield and your fingers on the front of the frame, then gently push the lens outward.



STEP 2: REMOVE SHIELD

Grip the wicket and pull the shield downward to disengage the lens.



STEP 3: ALIGN AND SEAT THE RX INSERT

Locate the center attachment clip on the RX Insert and align it with the mounting point located on the nose bridge.



STEP 4: SECURE THE RX INSERT

Tilt the RX Insert and fit the metal wicket into the RX Insert clip.



STEP 5: REINSTALL THE SHIELD BACK INTO THE FRAME

Hook the top edge of the shield into the frame. Press the shield inward from bottom until it clicks, to securely into place.



REMOVING THE RX INSERT

- Remove the shield by following Steps 1 & 2.
- Lift the RX Insert away from the metal wicket to disengage the RX Insert.
- Tilt the RX Insert forward and carefully remove it from the frame.
- Reinstall the shield by following Steps 4 & 5.