



Surge RX Insert for Oculus Quest Pro (Installation Instructions)

Thank you for choosing your custom prescription RX Insert. Follow the steps below to properly install the RX Insert into your Oculus Quest Pro eyewear for a secure fit and optimal visual performance.

STEP 1: CLEAN THE QUEST PRO LENSES

Using the microfiber cloth included in the package, gently wipe the inner lenses of the Oculus Quest Pro. Use light pressure. Do not press too hard.



STEP 2: ALIGN AND ATTACH THE FIRST INSERT

Hold the first insert up to its matching Oculus Quest Pro lens. Carefully align it so it sits centered over the lens, then gently press it into place. Gently press until it clicks into place.



STEP 3: ATTACH THE SECOND INSERT

Repeat the same process on the other side, align the second insert over its matching lens and gently press it into place.



STEP 4: CHECK THAT BOTH INSERTS ARE SECURE

Lightly wiggle each insert to confirm neither one is loose or shifting. Both should feel firm and stable with no movement before you put the headset on.



REMOVING THE RX INSERT

- Look at the inner lens area of the Oculus Quest Pro and find the outer edge of the prescription insert where it meets the lens housing.
- Using your fingertip or thumbnail, carefully apply light pressure to the edge of the insert and gently pop it away from the lens. Work slowly. Do not force it or use sharp objects.
- Place both inserts back into their protective case or original packaging to keep them safe and scratch-free.